

# INFORMATION SHEET



## HLP Medi-Temp Guide

This unit has **two** temperature sensors, one inbuilt inside the **unit (IN)** and another sensor on a **probe** at the end of a 2-metre cable (**OUT**). See *diagram below*:



Button	Function	How to use
<b>MAX / MIN</b>	Display maximum and minimum temperature	<ul style="list-style-type: none"><li>Press once to display max temperature</li><li>Press again to display min temperature</li><li>Press and HOLD to reset</li></ul>
<b>IN / OUT</b>	Switch display between "IN" and "OUT" temperatures	<ul style="list-style-type: none"><li>"IN" displays the <b>unit</b> temperature</li><li>"OUT" refers to the <b>probe</b> temperature</li><li><b>Display should be set to OUT</b></li></ul>
<b>°C/°F</b>	Change between Celsius and Fahrenheit	<ul style="list-style-type: none"><li>Click once to change between Celsius and Fahrenheit</li><li><b>NOTE:</b> Button is located on the <b>back of unit</b></li></ul>
<b>Alarm</b>	Turn alarm on and off	<ul style="list-style-type: none"><li>Press once to turn on alarm</li><li>Press again to turn off alarm</li></ul>
<b>Alarm</b>	Set alarm parameters	<ul style="list-style-type: none"><li>Parameters should be set at <b>2°C</b> (low) to <b>8°C</b> (high)</li><li>Press and HOLD alarm button</li><li>Set high alarm by pushing °C/°F button until it displays 8°C</li><li>Press alarm button again</li><li>Set low alarm by pushing °C/°F button until it displays 2°C</li><li>Press alarm button again to save</li></ul>

### Further information

- [Medi-Temp information sheet](#)
- [Medi-Temp instruction sheet](#)
- [YouTube Medi-Temp demonstration video](#)